

## Symbol-Legende Icon Legend

	Achtung Attention		Europäischer Bevollmächtigter European Authorized Representative
	Artikelnummer Article number		CE-Kennzeichnung CE marking
	einzelner Patient- mehrfach anwendbar single patient- multiple use		Übersetzung Translation
	Reinigung Cleaning		Vertriebspartner Distributor
	Hersteller Manufacturer		Importeur Importeur
	Nicht bleichen Do not bleach		Keine chemische Reinigung No chemical cleaning
	Nicht bügeln Do not iron		Nicht im Trockner trocknen Do not tumble-dry
	Umpacken Repackaging		

Qualität und Funktion

## Gebrauchsanweisung Instruction Manual

### Schaftadapter A10, A11, A13, A14 Socket Adapter A10, A11, A13, A14



Rev. 3-2023-04\_A10,A11,A13,A14\_101708



Uniprox GmbH & Co.KG  
H.-Heine-Str.4  
07937 Zeulenroda-Triebes

Tel. +49 (0) 36628-66-33 00  
Fax +49 (0) 36628-66-33 55  
E-Mail [info@uniprox.de](mailto:info@uniprox.de)





Please read the IFU carefully before fitting. Only correct usage will warrant the function.

## 1. Intended Use

4-Hole Adapters are the connection between a pyramid attachment or a pyramid receiver and 4-hole threat adapter to a modular prosthetic limb prosthesis.

## 2. Technical data

### 2.1 A10 Socket Adapter

- 4-Hole Adapter w / pyramid receiver fixed
- water resistant\*: A10-St, A10-Ti



Order No.	Material	Weight Limit	Weight	Article No.
A10-Al	Aluminum	< 75 kg	45 g	4 114 020 00 11 000
A10-St	Steel	< 125 kg	79 g	4 114 020 00 22 000
A10-Ti	Titanium	< 150 kg	51 g	4 114 020 00 33 000

\* with specially waterproof coated bolts; not included

### 2.2 A11 Socket Adapter

- 4-Hole Adapter w / pyramid rotatable receiver
- water resistant\*
- is supplied with a 4-hole spacer for rotatable socket adapter (E-A03)



Order No.	Material	Weight Limit	Weight	Article No.
A11-St	Steel	< 125 kg	123 g	4 114 030 00 22 000
A11-Ti	Titanium	< 150 kg	79 g	4 114 030 00 33 000

\* with specially waterproof coated bolts; not included

### 2.3 A13 Socket Adapter

- 4-Hole Adapter w / pyramid fixed
- water resistant\*: A13-St, A13-Ti, A13



Order No.	Material	Weight Limit	Weight	Article No.
A13-Al	Aluminum	< 75 kg	58 g	4 114 040 00 44 000
A13-St	Steel	< 100kg	94 g	4 114 040 00 22 000
A13-Ti	Titanium	< 150 kg	51 g	4 114 040 00 33 000
A13	Titanium, flat attachment surface	< 150 kg	50 g	4 114 040 03 33 000

\* with specially waterproof coated bolts; not included

### 2.4 A14 Socket Adapter

- 4-Hole Adapter w / rotatable pyramid
- water resistant\*
- A14-Ti: including 4-hole spacer for rotatable socket adapter (E-A03)



Order No.	Material	Weight Limit	Weight	Article No.
A14-St	Steel	< 100 kg	128 g	4 114 050 00 22 000
A14-Ti	Titanium	< 150 kg	85 g	4 114 050 00 33 000

\* with specially waterproof coated bolts; not included

## 3. Indications/ Contraindications

Indications:

- Amputation of lower extremities

Contraindications:

- not known

#### 4. Side effects

There are no known side effects.

#### 5. General Instructions



- This medical product is designed for: single patient, multiple use.
- Fitting/service of this medical device is only allowed by a certificated orthopedic professional.
- Improper modification or application to the product is not allowed. In case of non-observance, the function of the product may be impaired, so that product liability is excluded.
- Only approved screws may be used.
- The maximum weight loads are listed under "2. Technical Data". Additional loads must be taken into account when calculating the weight.

#### 6. Installation

The torque of the countersunk screw is 12 Nm.

Additional for A11 and A14-Ti:

Always use the attached spacer screw to guarantee the safe deadlock of the rotational toothing. The torque of the adjusting screw is also 12 Nm.



**The various screws used for assembly purposes must be secured with Loctite 222.**



**Waterproof adapters can be used, when the adapter crews have been changed for waterproof coated crews (E-A10, E-A11)! They are not included.**

#### 7. Maintenance



The adapters must be checked as part of regular prosthetic service/maintenance.

#### 8. CE-Conformity

The product satisfies the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council (MDR) and bears the CE mark. All major incidents related to the product needs to be informed to Uniprox and the competence European Authority.

#### 9. Warranty and Guarantee

We recommend to change/renew the adapter after 5 years of usage.

Warranty is provided under the terms of sale and supply of Uniprox GmbH & Co. KG provided that the above conditions are met.

#### 10. Storage and Disposal

The product is disposable with standard household garbage.

There are no storage recommendations.

#### Please direct any questions to:

Customer Service: +49 (0) 36628-66-33 70  
 Fax: +49 (0) 36628-66-33 77  
 E-mail: [export@uniprox.de](mailto:export@uniprox.de)