

Qualität und Funktion

Gebrauchsanweisung
Instruction Manual

Modular SACH foot adapter
E-F01, E-F02, E-F04



Rev.0-2021-06_E-F01,E-F02,E-F04_101700



Uniprox GmbH & Co.KG
H.-Heine-Str.4
07937 Zeulenroda-Triebes

Tel. +49 (0) 36628-66-33 00
Fax +49 (0) 36628-66-33 55
E-Mail info@uniprox.de





Please read the IFU carefully before fitting. Only correct usage will warrant the function.

1. Intended Use

The modular adapters E-F01, E-F02, E-F04 are the connection to the Sach feet F1 and F20.

2. Technical data

- connector: pyramid with a \varnothing 50 mm spherical segment
- incl. bolt assemblies (M10)
- incl. 2-component glue
- water resistant (E-F02, E-F04 only)

REF

Order No.	Weight limit	Description	Article No.
E-F01	< 75 kg	Aluminium 10M	4 101 019 99 11 010
E-F02	< 125 kg	Steel M10	4 101 019 99 22 010
E-F04	< 100 kg	Titanium M10	4 101 019 99 33 010

3. Indication/ Contraindication

Indication:

- Amputation of lower extremities

Contraindication:

- not known

4. Side effects

There are no known side effects.

5. General Instructions



- This medical product is designed for: single patient, multiple use.
- Fitting/service of this medical device is only allowed by a certificated orthopedic professional.
- Improper modification or application to the product is not allowed. In case of non-observance, the function of the product may be impaired, so that product liability is excluded.
- Only approved screws may be used.
- The maximum weight loads are listed under "2. Technical Data". Additional loads must be taken into account when calculating the weight.

6. Installation

Tighten the included M10 screw with a torque wrench to 30Nm.



Secure the screw with Loctite 243 when finishing the prosthesis.



Connect the SACH-Foot adapter to the SACH-Foot by using the included 2-component glue.



7. Maintenance

The socket attachment block must be checked as part of regular prosthetic service/maintenance.

8. CE-Conformity

The product satisfies the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council (MDR) and bears the CE mark. All major incidents related to the product needs to be informed to Uniprox and the competence European Authority.

9. Warranty and Guarantee

We recommend to change/renew the adapter after 5 years of usage.

Warranty is provided under the terms of sale and supply of Uniprox GmbH & Co. KG provided that the above conditions are met.

10. Storage and Disposal

The product is disposable with standard household garbage.

There are no storage recommendations.

Please direct any questions to:

Customer Service: + 49 (0) 36628-66-33 70

Fax: + 49 (0) 36628-66-33 77

E-mail: export@uniprox.de