Symbol-Legende Icon Legend

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7. CE-Conformity

The product satisfies the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council (MDR) and bears the CE mark. All major incidents related to the product needs to be informed to *Uniprox and/or Orthomerica Products* as well as to the competence European Authority.

8. Warranty

Warranty is provided under the terms of sale and supply of Uniprox GmbH & Co. KG provided that the above conditions are met.

9. Storage and Disposal

The orthosis has no special storage regulations. Please note the country-specific disposal.

- Deutsch -

- English -

Please direct any questions to:

Customer Service: + 49 (0) 36628-66-33 70
 Fax: + 49 (0) 36628-66-33 77
 E-mail: info@uniprox.de

5. General Instructions

- This medical product is designed for single patient, multiple use.
- Fitting/service of this medical device is only allowed by a certificated orthopedic professional.
- Wear the splint according to the instructions of your doctor.
- Always wear the night splint when you are resting for longer periods. The night splint was designed to be worn while sleeping.
- Only use the foam wedge provided after consulting your doctor. The wedge is pushed under the toes in order to make the sole of the foot turn more. It is pushed under the padded lining and secured to the splint with the existing Velcro fastening.

Note:

The higher side of the wedge must point towards the front edge of the splint.

- English -

- To improve wearing comfort, a sports sock should be worn directly above the foot.
- When wearing the splint the Velcro fastening should fit well, but not too tightly.
- Consult your doctor if your leg begins to tingle or becomes numb when wearing the splint.
- Do not discontinue treatment with the orthotic without consultation.
- Do not wear the orthotic if it irritates your skin.
- Do not put too much pressure on the foot.
- Never keep the orthotic in the vicinity of heat sources.

6. Cleaning

If necessary, clean the padded lining using only mild soap and water.

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Please read the IFU carefully before fitting. Only correct usage will warrant the function.

1. Intended Use

The OF08 night splint is used exclusively for the temporary immobilization of the upper and lower ankle joint following post-operative measures. It is primarily used when the patient is sleeping and is not suitable for daily use (constant walking).

2. Technical Data

The night splint is made of a thermoplastic deformable plastic and has a comfortable, removable, washable inner pad.

The adjustable extension straps ensure optimal adjustment to the patient.

Order No.	Shoe Size	Size	Side	Article No.
OF08-1	36 – 39	S	universal	4 532 010 01 00 000
OF08-2	39 – 41,5	M	universal	4 532 010 02 00 000
OF08-3	41,5 - 45	L	universal	4 532 010 03 00 000

- English -

3. Indications/ Contraindications

Indications:

- For temporary immobilization of the upper and lower ankle joint
- After surgical interventions
- For pointed foot prophylaxis/treatment

Contraindications:

- None

4. Side effects

Where the advice provided is followed there are no specific risks to be anticipated in wearing the OF08 night splint.