Gebruiksaanweisung für den Techniker
Instructions for use for technicians
Mode d'emploi pour le technicien
Istruzioni per l'uso per il tecnico
Gebruiksaanwijzing voor de technicus

Schaftadapter A10, A11, A13, A14

Socket Adapter A10, A11, A13, A14

Version: 5, Stand: 06.11.2025

Uniprox GmbH & Co. KG
H.-Heine-Str. 4
07937 Zeulenroda-Triebes

Tel. +49 (0) 36628-66-33 00
Fax +49 (0) 36628-66-33 55
E-Mail info@uniprox.de

Symbole/ Icons/ Symboles/ Simboli/ Symbolen	3
DE - Gebrauchsanweisung.....	4
EN - Instruction Manual	8
FR - Mode d'emploi	12
IT - Istruzioni per l'uso	16
NL - Gebruiksaanwijzing	20

Symbole/ Icons/ Symboles/ Simboli/ Symbolen

Hersteller
Manufacturer
Fabricant
Produttore
Fabrikant

Herstellungsdatum
Date of manufacture
Date de fabrication
Data di produzione
Productiedatum

Artikelnummer
Article number
Numéro d'article
Numero articolo
Artikelnummer

Achtung
Attention
A noter
Attenzione
Achtung

einzelner Patient - mehrfach anwendbar
single patient - multiple use
patient individuel - utilisable plusieurs fois
singolo paziente - utilizzabile più volte
één patiënt - meerdere keren te gebruiken

Fertigungslosnummer, Charge
Production lot number, batch
Numéro de lot de fabrication, lot
Numero di lotto di produzione, partita
Productienummer, batch

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EN - Instruction Manual

Please read the IFU carefully before fitting.
Only correct usage will warrant the function.
The instructions for use are not intended for distribution to patients!

1. Intended Use

Adapters are load-bearing components within a prosthesis.

2. Technical data

2.1 A10 Socket Adapter

- 4-Hole Adapter w / pyramid receiver fixed
- water resistant*: A10-St, A10-Ti

REF

Order No.	Material	Weight Limit	Weight	Article No.
A10-Al	Aluminum	< 75 kg	45 g	4 114 020 00 11 000
A10-St	Steel	< 125 kg	79 g	4 114 020 00 22 000
A10-Ti	Titanium	< 150 kg	51 g	4 114 020 00 33 000

* with especially waterproof coated bolts; not included

2.2 A11 Socket Adapter

- 4-Hole Adapter w / pyramid rotatable receiver
- water resistant*
- is supplied with a 4-hole spacer for rotatable socket adapter (E-A03)

REF

Order No.	Material	Weight Limit	Weight	Article No.
A11-St	Steel	< 125 kg	123 g	4 114 030 00 22 000
A11-Ti	Titanium	< 150 kg	79 g	4 114 030 00 33 000

* with especially waterproof coated bolts; not included

2.3 A13 Socket Adapter

- 4-Hole Adapter w / pyramid fixed
- water resistant*: A13-St, A13-Ti, A13

REF

Order No.	Material	Weight Limit	Weight	Article No.
A13-Al	Aluminum	< 75 kg	58 g	4 114 040 00 44 000
A13-St	Steel	< 100kg	94 g	4 114 040 00 22 000
A13-Ti	Titanium	< 150 kg	51 g	4 114 040 00 33 000
A13	Titanium, flat attachment surface	< 150 kg	50 g	4 114 040 03 33 000

* with especially waterproof coated bolts; not included

2.4 A14 Socket Adapter

- 4-Hole Adapter w / rotatable pyramid
- water resistant*
- A14-Ti: including 4-hole spacer for rotatable socket adapter (E-A03)

REF

Order No.	Material	Weight Limit	Weight	Article No.
A14-St	Steel	< 100 kg	128 g	4 114 050 00 22 000
A14-Ti	Titanium	< 150 kg	85 g	4 114 050 00 33 000

* with especially waterproof coated bolts; not included

2.5 Individual and spare parts

4-Hole spacer for rotatable socket adapter (A11, A14)

REF

Order No.	Material	Units p. package	Weight Limit	Weight	Article No.
E-A03	Steel	1	< 150 kg	11 g	4 114 029 00 00 000

4-Hole spacer set

REF

Order No.	Material	Units p. package	Weight Limit	Weight	Article No.
E-A06	Aluminium	3	< 150 kg	32 g	4 620 009 00 34 000

Hex socket head screw

REF

Order No.	Thread	Length	Units p. package	Article No.
M28-12	M8	12 mm	10	4 226 050 00 00 012
M28-16	M8	16 mm	10	4 226 050 00 00 016

Hex socket head screw with waterproof coating

REF

Order No.	Thread	Length	Units p. package	Article No.
E-A11-12	M8	12 mm	4	4 113 018 11 20 612

Hexagon socket countersunk head screws with waterproof coating

REF

Order No.	Thread	Length	Units p. package	Article No.
E-A10-20	M6	20 mm	4	4 114 028 10 20 020
E-A10-25	M6	25 mm	4	4 114 028 10 20 025

3. Indications/ Contraindications

Indications:

- Leg amputation
- Dysmelia

Contraindications:

- none

4. Side effects

There are no known side effects.

5. General safety instructions

- This medical product is designed for: single patient, multiple use.
- Fitting/ service of this medical device is only allowed by a certificated orthopedic professional.
- The patient must be instructed by the technician on correct use and informed about possible side effects/ residual risks.
- Improper modification or application to the product is not allowed. In case of non-observance, the function of the product may be impaired, so that product liability is excluded.
- Only approved screws may be used.
- The maximum weight loads are listed under "2. Technical Data". Additional loads must be taken into account when calculating the weight.

6. Residual risks

Despite proper installation and use of the fitting parts in a prosthesis, adverse circumstances can lead to insecurity (e.g. stumbling, loss of balance, falling).

7. Installation

The torque of the countersunk screw is 12 Nm.

Additional for A11 and A14-Ti:

Always use the attached spacer screw to guarantee the safe deadlock of the rotational tothing. The torque of the adjusting screw is also 12 Nm.

The various screws used for assembly purposes must be secured with Loctite® 222.

Waterproof adapters can be used when the adapter screws have been changed for waterproof coated screws (E-A10, E-A11)!

They are not included.

8. Maintenance and Cleaning

The adapters must be checked as part of regular prosthetic service/ maintenance.

Soap and hand warm water

Do not use aggressive solvents for cleaning.

9. CE-Conformity

The product satisfies the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council (MDR) and bears the CE mark. All major incidents related to the product need to be informed to Uniprox and the competence European Authority.

10. Warranty and Guarantee

We recommend to change/ renew the adapter after 5 years of usage.

A warranty is provided under the terms of sales and supply of Uniprox GmbH & Co. KG provided that the above conditions are met.

11. Storage and Disposal

This product has no special storage regulations.

Please note the country-specific disposal.

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