Gebruiksaanweisung für den Techniker
Instructions for use for technicians
Mode d'emploi pour le technicien
Istruzioni per l'uso per il tecnico
Gebruiksaanwijzing voor de technicus

Schraub-Adaption A22, A23, A25

Threaded Adapters A22, A23, A25

M36-Adapter A87

M36-Adapter A87

Version: 3, Stand: 06.11.2025

Uniprox GmbH & Co. KG
H.-Heine-Str. 4
07937 Zeulenroda-Triebes

Tel. +49 (0) 36628-66-33 00
Fax +49 (0) 36628-66-33 55
E-Mail info@uniprox.de

Symbole/ Icons/ Symboles/ Simboli/ Symbolen	3
DE - Gebrauchsanweisung.....	4
EN - Instruction Manual	7
FR - Mode d'emploi	10
IT - Istruzioni per l'uso	13
NL - Gebruiksaanwijzing	16

Symbole/ Icons/ Symboles/ Simboli/ Symbolen

Hersteller
Manufacturer
Fabricant
Produttore
Fabrikant

Herstellungsdatum
Date of manufacture
Date de fabrication
Data di produzione
Productiedatum

Artikelnummer
Article number
Numéro d'article
Numero articolo
Artikelnummer

Achtung
Attention
A noter
Attenzione
Achtung

einzelner Patient - mehrfach anwendbar
single patient - multiple use
patient individuel - utilisable plusieurs fois
singolo paziente - utilizzabile più volte
één patiënt - meerdere keren te gebruiken

Fertigungslosnummer, Charge
Production lot number, batch
Numéro de lot de fabrication, lot
Numero di lotto di produzione, partita
Productienummer, batch

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EN - Instruction Manual

Please read the IFU carefully before fitting.
Only correct usage will warrant the function.
The instructions for use are not intended for distribution to patients!

1. Intended Use

Adapters are load-bearing components within a prosthesis.

2. Technical data

2.1 A22 Threaded Adapters

- M36 x 1,5 Thread and Pyramid
- Waterproof *

REF

Order No.	Material	Weight Limit	Weight	Article No.
A22-St	Steel	< 125 kg	70 g	4 114 090 99 22 000
A22-Ti	Titanium	< 150 kg	40 g	4 114 090 99 33 000

2.2 A23 Threaded Adapters

- M36 x 1,5 Thread and pyramid receiver
- waterproof*
- Pyramid fixture has a standard ball section with Ø 50 mm

REF

Order No.	Material	Weight Limit	Weight	Article No.
A23-St	Steel	< 150 kg	78 g	4 114 090 77 22 000
A23-Ti	Titanium	< 100 kg	51 g	4 114 090 77 33 000

* with specially waterproof coated bolts; not included

2.3 A25 Threaded Adapters

- M36 x 1,5 thread and tube clamp 30 mm
- weight limit: 100 kg

REF

Order No.	Material	Weight	Article No.
A25-Ti	Titanium	69 g	4 114 090 88 33 000

2.4 A87 M36-Adapter

- M36 x 1,5 outside thread and pyramid receiver
- weight limit: 150 kg
- waterproof*

REF

Order No.	Material	Weight	Article No.
A87	Steel	55 g	4 620 040 00 00 000

* with specially waterproof coated bolts; not included

2.5 Individual and spare parts

Hex socket head screw:

REF

Order No.	Thread	Length	Units p. package	Article No.
M28-12	M8	12 mm	10	4 226 050 00 00 012
M28-16	M8	16 mm	10	4 226 050 00 00 016

Hex socket head screw with waterproof coating:

REF

Order No.	Thread	Length	Units p. package	Article No.
E-A11-12	M8	12 mm	4	4 113 018 11 20 612

Hexagon socket head cap screw:

REF

Order No.	Thread	Length	Units p. package	Article No.
M33	M5	18 mm	10	4 226 030 05 00 018

Hexagon socket head cap screw with waterproof coating:

REF

Order No.	Thread	Length	Units p. package	Article No.
E-A12-20	M5	20 mm	1	4 113 018 12 20 520

3. Indications/ Contraindications

Indications:

- Leg amputation
- Dysmelia

Contraindications:

- none

4. Side effects

There are no known side effects.

5. General safety instructions

- This medical product is designed for single patient multiple use.
- Fitting/ service of this medical device is only allowed by a certificated orthopedic professional.
- The patient must be instructed by the technician on correct use and informed about possible side effects/ residual risks.
- Improper modification or application to the product is not allowed.
- In case of non-observance, the function of the product may be impaired, so that product liability is excluded.
- The maximum weight loads are listed under "2. Technical Data". Additional loads must be taken into account when calculating the weight

6. Residual risks

Despite proper installation and use of the fitting parts in a prosthesis, adverse circumstances can lead to insecurity (e.g. stumbling, loss of balance, falling).

7. Instructions of Use - Headless Screws

By using the adjusting screws M28-12 or M28-16, static corrections are possible at any time. If a module must be replaced or removed the previously adjusted position is maintained if only two adjacent adjusting screws (not opposite), the ones screwed in at the deepest, are removed.

If the adjusting screws are too long (damage of the foam cover) or too short (reduced stability) these must be replaced with appropriate ones.

Use a torque wrench to tighten the adjusting screws.

Tightening torque: 12 Nm.

Secure the various screws, used for assembly, with Loctite® 222.

8. Maintenance and Cleaning

The adapters must be checked as part of regular prosthetic service/ maintenance.

Soap and hand warm water

Do not use aggressive solvents for cleaning.

9. CE-Conformity

The product satisfies the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council (MDR) and bears the CE mark. All major incidents related to the product need to be informed to Uniprox and the competence European Authority.

10. Warranty and Guarantee

We recommend to change/ renew the adapter after 5 years of usage. A warranty is provided under the terms of sales and supply of Uniprox GmbH & Co. KG provided that the above conditions are met.

11. Storage and Disposal

This product has no special storage regulations. Please note the country-specific disposal.

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