Gebruiksaanweisung für den Techniker
Instructions for use for technicians
Mode d'emploi pour le technicien
Istruzioni per l'uso per il tecnico
Gebruiksaanwijzing voor de technicus

Anschlussplatte A26, A27

Attachment Plate A26, A27

Eingussscheibe A28, A29

Lamination Attachment Plate A28, A29

Version: 3, Stand: 06.11.2025

Uniprox GmbH & Co. KG
H.-Heine-Str. 4
07937 Zeulenroda-Triebes

Tel.
Fax
E-Mail

+49 (0) 36628-66-33 00
+49 (0) 36628-66-33 55
info@uniprox.de

Symbole/ Icons/ Symboles/ Simboli/ Symbolen	3
DE - Gebrauchsanweisung.....	4
EN - Instruction Manual	7
FR - Mode d'emploi	10
IT - Istruzioni per l'uso	13
NL - Gebruiksaanwijzing	16

Symbole/ Icons/ Symboles/ Simboli/ Symbolen

Hersteller
Manufacturer
Fabricant
Produttore
Fabrikant

Herstellungsdatum
Date of manufacture
Date de fabrication
Data di produzione
Productiedatum

Artikelnummer
Article number
Numéro d'article
Numero articolo
Artikelnummer

Achtung
Attention
A noter
Attenzione
Achtung

einzelner Patient - mehrfach anwendbar
single patient - multiple use
patient individuel - utilisable plusieurs fois
singolo paziente - utilizzabile più volte
één patiënt - meerdere keren te gebruiken

Fertigungslosnummer, Charge
Production lot number, batch
Numéro de lot de fabrication, lot
Numero di lotto di produzione, partita
Productienummer, batch

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EN - Instruction Manual

Please read the IFU carefully before fitting.
Only correct usage will warrant the function.
The instructions for use are not intended for distribution to patients!

1. Intended Use

Adapters are load-bearing components within a prosthesis.

2. Technical data

2.1 A26 Attachment Plate

- for transtibial sockets
- diameter: 64 mm, Height: 9 mm, back side sandblasted
- incl. screws and Dummy
- weight limit: 100 kg

REF

Order No.	Material	Weight	Article No.
A26-AL	Aluminum	74 g	4 115 010 00 00 064

2.2 A27 Attachment Plate

- for Transfemoral Sockets
- diameter: 84 mm, height: 6 mm, back side sandblasted
- incl. screws and Dummy

REF

Order No.	Material	Weight Limit	Weight	Article No.
A27-AL1	Aluminum, centered	< 150 kg	91 g	4 115 010 00 00 084
A27-AL2	Aluminum, 8 mm posterior off-set	< 100 kg	77 g	4 115 010 01 00 084
A27-AL3	Aluminum, 12 mm posterior off-set	< 100 kg	68 g	4 115 010 02 00 084

2.3 A28 Lamination Attachment Plate

- for direct fitting to Locking devices
- Diameter: 70 mm, Height: 7 mm
- incl. screws, Dummy not included
- Dummy (E-A05) needs to be ordered separately E-A05
- water resistant*
- weight limit: 150 kg

REF

Order No.	Material	Weight	Article No.
A28-AL	Aluminum	55 g	4 147 009 21 00 000

* with especially waterproof coated bolts; not included

2.4 A29 Lamination Attachment Plate

- for direct fitting to Locking devices
- measures: 52x52 mm, Height: 7 mm
- incl. screws, exclusive
- Dummy (E-A05) needs to be ordered separately
- weight limit: 100 kg

REF

Order No.	Material	Weight	Article No.
A29-AL	Aluminum	55 g	4 147 009 01 00 000

2.5 Individual and spare parts

REF

Order-No.	Description	Units p. package	Article No.
E-A05	Lamination Dummy for attachment plates	1	4 114 110 00 00 001
E-A10-20	Hexagon socket countersunk head screws with waterproof coating (for lamination plate A28-AL)	4	4 114 028 10 20 020

3. Indications/ Contraindications

Indications:

- Leg amputation
- Dysmelia

Contraindications:

- none

4. Side effects

There are no known side effects.

5. General safety instructions

- This medical product is designed for: single patient, multiple use.
- Fitting/ service of this medical device is only allowed by a certificated orthopedic professional.
- The patient must be instructed by the technician on correct use and informed about possible side effects/ residual risks.
- Improper modification or application to the product is not allowed. In case of non-observance, the function of the product may be impaired, so that product liability is excluded.
- Only approved screws may be used.
- The maximum weight loads are listed under "2. Technical Data". Additional loads must be taken into account when calculating the weight.

6. Residual risks

Despite proper installation and use of the fitting parts in a prosthesis, adverse circumstances can lead to insecurity (e.g. stumbling, loss of balance, falling).

7. Installation

Depending on the socket position, the lamination attachment plate needs to be glued to the rigid foam. Grind the foam below the lamination smaller than the diameter of the lamination attachment plate itself, to achieve adequate support in the laminate. Depending on the activity and the weight of the user there must be enough laminate for pouring. Tie the laminate carefully in the groove. Grind/ sand the laminate up to the dummy only.

The torque of the countersunk screw is 12 Nm.

Secure the various assembling screws with Loctite® 222.

7.1 Lamination Attachment Plate A28 and A29

Attach the plate to the locking device (ShL1, ShL20 or ShL3A-2) with the included screws. Depending on the activity and the weight of the user there must be enough laminate for pouring. Tie the laminate carefully in the groove. Avoid direct contact between aluminum and carbon.

Aluminium + Carbon + Moisture = Galvanic Corrosion!

Grind/ sand the laminate up to the dummy only.

The torque setting of the countersunk screw is 12 Nm.

Secure the various assembling screws with Loctite® 222.

8. Maintenance and Cleaning

The adapters must be checked as part of regular prosthetic service/ maintenance.

Soap and hand warm water

Do not use aggressive solvents for cleaning.

9. CE-Conformity

The product satisfies the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council (MDR) and bears the CE mark. All major incidents related to the product need to be informed to Uniprox and the competence European Authority.

10. Warranty and Guarantee

We recommend to change/ renew the adapter after 5 years of usage.

A warranty is provided under the terms of sales and supply of Uniprox GmbH & Co. KG provided that the above conditions are met.

11. Storage and Disposal

This product has no special storage regulations. Please note the country-specific disposal.

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