Gebruiksaanweisung für den Techniker
Instructions for use for technicians
Mode d'emploi pour le technicien
Istruzioni per l'uso per il tecnico
Gebruiksaanwijzing voor de technicus

Rohradapter A3, A4

Tube Adapter A3, A4

Prothesenrohr R1

Pylon R1

Version: 7, Stand: 06.11.2025

Uniprox GmbH & Co. KG
H.-Heine-Str. 4
07937 Zeulenroda-Triebes

Tel.
Fax
E-Mail

+49 (0) 36628-66-33 00
+49 (0) 36628-66-33 55
info@uniprox.de

Symbole/ Icons/ Symboles/ Simboli/ Symbolen	3
DE - Gebrauchsanweisung.....	4
EN - Instruction Manual	7
FR - Mode d'emploi	10
IT - Istruzioni per l'uso	13
NL - Gebruiksaanwijzing	16

Symbole/ Icons/ Symboles/ Simboli/ Symbolen

	Hersteller Manufacturer Fabricant Produttore Fabrikant		Herstellungsdatum Date of manufacture Date de fabrication Data di produzione Productiedatum
	Artikelnummer Article number Numéro d'article Numero articolo Artikelnummer		Achtung Attention A noter Attenzione Achtung
	einzelner Patient - mehrfach anwendbar single patient - multiple use patient individuel - utilisable plusieurs fois singolo paziente - utilizzabile più volte één patiënt - meerdere keren te gebruiken		Fertigungslosnummer, Charge Production lot number, batch Numéro de lot de fabrication, lot Numero di lotto di produzione, partita Productienummer, batch
	CE-Kennzeichnung CE marking Marquage CE Marcatura CE CE-markering		Medizinprodukt Medical device Dispositif médical Dispositivo medico Medisch hulpmiddel
	Eindeutige Gerätekenung Unique device identifier Identifiant unique de l'appareil Identificazione univoca del dispositivo Unieke apparaatidentificatie		Gebrauchsanweisung Instructions for use Mode d'emploi Istruzioni per l'uso Gebruiksaanwijzing
	Reinigung Cleaning Cleaning Pulizia Reiniging		Prothese Prosthesis Prothèse Protesi Prothese

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EN - Instruction Manual

Please read the IFU carefully before fitting.
Only correct usage will warrant the function.
The instructions for use are not intended for distribution to patients!

1. Intended Use

Adapters are load-bearing components within a prosthesis.

2. Technical data

2.1 A3 Tube Adapter

- for Ø 30 mm tubes
- weight limit: 100 kg

REF

Order No.	Material	Lenght	Weight	Article No.
A3-Al200	Aluminum	200 mm	160 g	4 113 051 30 11 200
A3-Al400	Aluminum	400 mm	270 g	4 113 051 30 11 400
A3-St200	Steel	200 mm	210 g	4 113 051 30 22 200
A3-St400	Steel	400 mm	320 g	4 113 051 30 22 400
A3-Ti200	Titanium	200 mm	170 g	4 113 051 30 33 200
A3-Ti400	Titanium	400 mm	280 g	4 113 051 30 33 400

2.2 A4 Tube Adapter

- for Ø 30 mm tubes
- weight limit: 100 kg

REF

Order No.	Material	Lenght	Weight	Article No.
A4-St200	Steel	200 mm	210 g	4 113 050 34 22 200
A4-St400	Steel	400 mm	320 g	4 113 050 34 22 400
A4-Ti400	Titanium	400 mm	280 g	4 113 050 34 33 400

2.3 Pylon R1

- for Ø 30 mm tubes
- weight limit: 100 kg

REF

Order No.	Material	Lenght	Weight	Article No.
R1-Al400	Aluminum	400 mm	210 g	4 113 070 30 11 400

2.4 Individual and spare parts

Hex socket head screw:

REF

Order No.	Thread	Lenght	Units p. package	Artikel No.
M28-12	M8	12 mm	10	4 226 050 00 00 012
M28-16	M8	16 mm	10	4 226 050 00 00 016

3. Indications/ Contraindications

Indications:

- Leg amputation
- Dysmelia

Contraindications:

- none

4. Side effects

There are no known side effects.

5. General safety instructions

- This medical product is designed for single patient, multiple use.
- Fitting/ service of this medical device is only allowed by a certificated orthopedic professional.
- The patient must be instructed by the technician on correct use and informed about possible side effects/ residual risks.
- Improper modification or application to the product is not allowed. In case of non-observance, the function of the product may be impaired, so that product liability is excluded.
- The maximum weight loads are listed under "2. Technical Data". Additional loads must be considered when calculating the weight.
- These adapters are not designed for use in water.

6. Residual risks

Despite proper installation and use of the fitting parts in a prosthesis, adverse circumstances can lead to insecurity (e.g. stumbling, loss of balance, falling).

7. Installation

Shorten the tube adapter to the appropriate length according to the user's preferred measurements.

Do not use a bench vise to fix the tube clamp or tube adapter. Be certain that the cut surface and the longitudinal axis of the tube are rectangular. Smoothen the area of the tube cut on the in- and outside.

Insert the tube into the clamp adapter until the tube reaches the bottom of the clamp. Fix the tube in the clamp adapter by tightening the clamp screw with a torque wrench.

Always position the tube clamp adapter proximally to the tube adapter.

It is not permitted to pull out the tube from the tube clamp adapter to adjust the length.

7.1 Instructions of Use - Headless Screws

By using the adjusting screws M28-12 or M28-16, static corrections are possible at any time. If a module must be replaced or removed the previously adjusted position is maintained if only two adjacent adjusting screws (not opposite), the ones screwed in at the deepest, are removed.

If the adjusting screws are too long (damage of the foam cover) or too short (reduced stability) these must be replaced with appropriate ones.

Use a torque wrench to tighten the adjusting screws.

Tightening torque: 12 Nm.

Secure the various screws, used for assembly, with Loctite® 222.

8. Maintenance and Cleaning

The adapters must be checked as part of regular prosthetic service/ maintenance.

Soap and hand warm water

Do not use aggressive solvents for cleaning.

9. CE-Conformity

The product satisfies the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council (MDR) and bears the CE mark. All major incidents related to the product need to be informed to Uniprox and the competence European Authority.

10. Warranty and Guarantee

We recommend to change/ renew the adapter after 5 years of usage.

A warranty is provided under the terms of sales and supply of Uniprox GmbH & Co. KG provided that the above conditions are met.

11. Storage and Disposal

This product has no special storage regulations. Please note the country-specific disposal.

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