Gebrauchsanweisung für den Techniker
Instructions for use for technicians
Mode d'emploi pour le technicien
Istruzioni per l'uso per il tecnico
Gebruiksaanwijzing voor de technicus

Schaftadapter A6, A8, A9, A15, A17, A18, A19, A20, A21

Socket adapter A6, A8, A9, A15, A17, A18, A19, A20, A21

Knieexartikulationsadapter A86

Knee disarticulation adapter A86

Version: 6, Stand: 06.11.2025

Uniprox GmbH & Co. KG
H.-Heine-Str. 4
07937 Zeulenroda-Triebes

Tel. +49 (0) 36628-66-33 00
Fax +49 (0) 36628-66-33 55
E-Mail info@uniprox.de

Symbole/ Icons/ Symboles/ Simboli/ Symbolen	3
DE - Gebrauchsanweisung.....	4
EN - Instruction Manual	9
FR - Mode d'emploi	14
IT - Istruzioni per l'uso	19
NL - Gebruiksaanwijzing.....	24

Symbole/ Icons/ Symboles/ Simboli/ Symbolen

Hersteller
Manufacturer
Fabricant
Produttore
Fabrikant

Herstellungsdatum
Date of manufacture
Date de fabrication
Data di produzione
Productiedatum

Artikelnummer
Article number
Numéro d'article
Numero articolo
Artikelnummer

Achtung
Attention
A noter
Attenzione
Achtung

einzelner Patient - mehrfach anwendbar
single patient - multiple use
patient individuel - utilisable plusieurs fois
singolo paziente - utilizzabile più volte
één patiënt - meerdere keren te gebruiken

Fertigungslosnummer, Charge
Production lot number, batch
Numéro de lot de fabrication, lot
Numero di lotto di produzione, partita
Productienummer, batch

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EN - Instruction Manual

Please read the IFU carefully before fitting.
Only correct usage will warrant the function.
The instructions for use are not intended for distribution to patients!

1. Intended Use

Adapters are load-bearing components within a prosthesis.

2. Technical data

2.1 A6 Socket Adapter

- lamination anchor, incl. dummy
- water resistant: A6-St, A6-Ti

REF

Order No.	Material	Weight Limit	Weight	Article No.
A6-Al	Aluminum	< 75 kg	62 g	4 114 010 00 11 000
A6-St	Steel	< 100 kg	87 g	4 114 010 00 22 000
A6-Ti	Titanium	< 100 kg	53 g	4 114 010 00 33 000

2.2 A8, A9 Socket Adapter

- lamination anchor for direct fixation to locking systems
- incl. screws and dummy

REF

Order No.	Material	Weight Limit	Weight	Article No.
A8-Ti	Titanium	< 100 kg	66 g	4 147 019 19 00 000
A9-Ti	Titanium	< 100 kg	69 g	4 147 009 20 00 000

2.3 A15, A17, A18 Socket Adapter

- A15: lamination anchor with pyramid fixed
- A17, A18: lamination anchor with inner thread (M36 x 1,5) incl. rotatable pyramid and lamination dummy
- water resistant (with especially waterproof coated screws; not included in the delivery!)

REF

Order No.	Material	Weight Limit	Weight	Article No.
A15-St	Steel, 3-arm long	< 125 kg	110 g	4 114 100 01 22 000
A17-St	Steel, 3-arm long	< 125 kg	167 g	4 114 070 99 22 000
A18-St	Steel, 3-arm long	< 125 kg	135 g	4 114 070 77 22 000

2.4 A19, A20, A21 Socket Adapter

- 3-arm lamination anchor w/thread M36x1,5 incl. Dummy
- water resistant (with especially waterproof coated screws; not included in the delivery!)

REF

Order No.	Material	Weight Limit	Weight	Article No.
A19-St	Steel, 3-Arm long	< 125 kg	110 g	4 114 080 01 22 000
A20-St	Steel, 4-arm long	< 150 kg	120 g	4 114 080 02 22 000
A21-St	Steel, 4-arm short	< 125 kg	91 g	4 114 080 03 22 000

2.5 A86 Knee Disarticulation Anchor

- lamination anchor w / thread M36 × 1,5 incl. Dummy
- water resistant (with especially waterproof coated screws; not included in the delivery!)

REF

Order No.	Material	Weight Limit	Weight	Article No.
A86	Steel	< 125 kg	170 g	4 114 080 06 22 000

2.6 Individual and spare parts

REF

Order No.	Description	Article No.
E-A02	Lamination Dummy made of special rubber	4 114 019 01 00 000

Outside thread M36 x 1,5

REF

Order No.	Material	Article No.
E-A04	Steel	4 114 110 00 00 000

Hex socket head screw (for A17)

REF

Order No.	Model	Lenght	Packaging Unit	Article No.
M28-12	Steel	M8 x 12	10	4 226 050 00 00 012
M28-16	Steel	M8 x 16	10	4 226 050 00 00 016
E-A11-12	Steel with waterproof coating	M8 x 12	4	4 113 018 11 20 612

Cylinder screw with hexagon socket

REF

Order No.	Model	Lenght	Packaging Unit	Article No.
M33	Steel	M5 x 18	10	4 226 030 05 00 018
E-A12-20	Steel with waterproof coating	M5 x 20	1	4 113 018 12 20 520

3. Indications/ Contraindications

Indications:

- Leg amputation
- Dysmelia

Contraindications:

- none

4. Side effects

There are no known side effects.

5. General safety instructions

- This medical product is designed for single patient, multiple use.
- Fitting/ service of this medical device is only allowed by a certificated orthopedic professional.
- The patient must be instructed by the technician on correct use and informed about possible side effects/ residual risks.
- Improper modification or application to the product is not allowed. In case of non-observance, the function of the product may be impaired, so that product liability is excluded.
- Only approved screws may be used.
- The maximum weight loads are listed under "2. Technical Data". Additional loads must be taken into account when calculating the weight.
- In case of insufficient laminate reinforcement, the Socket Adaptor A6, A8, A9, A15 could break out from the prosthetic socket.
- Pay attention to sufficient form-fit (A8).

6. Residual risks

Despite proper installation and use of the fitting parts in a prosthesis, adverse circumstances can lead to insecurity (e.g. stumbling, loss of balance, falling).

7. Installation

7.1 A6, A8, A9, A15 Socket Adapter

- Bending the Socket Adapter A6 and A9 could result in crackings in the titanium.
- Seal the male pyramid and the distal holes with plastilina.
- Before placing the socket adaptor put some layers of reinforcement material over the plaster model.
- Then position socket adaptor on the model and tie off the following layers of reinforcement in the groove of the adapter with carbon fibre roving.
- Fix the enclosed Lamination Dummy E-A02 on the socket adaptor.
- The Lamination Dummy E-A02 covers and protects the pyramid during lamination procedure and provides a precise fitting brim of the laminate.
- During hardening process of A8 and A9, the lamination around the adapter must be cooled.
- After lamination expose the attachment side and holes of the adaptor. Do not damage the socket adapter.

7.2 A17-A21 Socket Adapter

7.2.1 A17, A19-A21

Screw the E-A04 bending protection in the thread M36x1,5 before bending.

For bending the adapter only use bending iron with rounded tins.

The bore-hole area should not be round off too strongly as failure and/ or Rupture of the material could occur.

7.2.2 A17-A21

Before lamination remove pyramid receptacle from the socket adapter. Protect the thread and the clamping of the socket adapter from resin with e.g. plastilina. Before placing the socket adapter put some layers of reinforcement material over the plaster model. Then position socket adapter on the model and tie off the following layers of reinforcement before the thread. Expose thread after lamination, insert pyramid receptacle and tighten clamp screw with Torque Wrench.

Torque: 10 Nm

7.2.3 A17

Handling of the Headless Screws

By using the adjusting screws M28-12 or M28-16, static corrections are possible at any time. If a module must be replaced or removed the previously adjusted position is maintained if only two adjacent adjusting screws (not opposite), the ones screwed in at the deepest, are removed.

If the adjusting screws are too long (damage of the foam cover) or too short (reduced stability) these must be replaced with appropriate ones.

Use a torque wrench to tighten the adjusting screws.

Tightening torque: 12 Nm.

Secure the various screws, used for assembly, with Loctite® 222.

7.3 A86 Knee Disarticulation Anchor

Screw the E-A04 bending protection in the thread M36x1,5 before bending.

For bending the adapter only use bending iron with rounded tins. The bore-hole area should not be round off too strongly as failure and/ or Rupture of the material could occur.

Use the dummy included in the scope of delivery to protect the thread from penetrating resin with plasticine. The adapter A86 can also be secured with screws for assembly. These screws are to be held in the definitive socket. When laminating, glass or carbon fiber threads are pulled through the holes in the adapter. These serve as a firm anchorage in the laminate.

The technician must ensure that the processing and reinforcement in the socket are adapted to the weight and mobility of the patient.

Attention: When the knee joint is fully flexed, there must be no contact between the socket and the knee joint. Defect of the prosthesis components possible.

Secure the clamp screw with Loctite® 222 and tighten with 8 Nm.

8. Maintenance and Cleaning

The adapters must be checked as part of regular prosthetic service/ maintenance.

Soap and hand warm water

Do not use aggressive solvents for cleaning.

9. CE-Conformity

The product satisfies the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council (MDR) and bears the CE mark. All major incidents related to the product needs to be informed to Uniprox and the competence European Authority.

10. Warranty and Guarantee

We recommend to change/ renew the adapter after 5 years of usage.

A warranty is provided under the terms of sales and supply of Uniprox GmbH & Co. KG provided that the above conditions are met.

11. Storage and Disposal

This product has no special storage regulations.

Please note the country-specific disposal.

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