Gebrauchsanweisung für den Techniker  
*Instructions for use for technicians*  
Mode d'emploi pour le technicien  
*Istruzioni per l'uso per il tecnico*  
Gebruiksaanwijzing voor de technicus

**Drehadapter TA-1, TA-2**  
*Rotation Adapters TA-1, TA-2*

Version: 6, Stand: 20.02.2026

Uniprox GmbH & Co. KG  
H.-Heine-Str. 4  
DE-07937 Zeulenroda-Triebes

Tel. +49 (0) 36628-66-33 00  
Fax +49 (0) 36628-66-33 55  
E-Mail [info@uniprox.de](mailto:info@uniprox.de)

Symbole/ Icons/ Symboles/ Simboli/ Symbolen .....	3
DE - Gebrauchsanweisung.....	4
EN - Instruction Manual .....	7
FR - Mode d'emploi .....	10
IT - Istruzioni per l'uso .....	13
NL - Gebruiksaanwijzing.....	16

## Symbole/ Icons/ Symboles/ Simboli/ Symbolen

	<p>Hersteller            Manufacturer            Fabricant            Produttore            Fabrikant</p>		<p>Herstellungsdatum            Date of manufacture            Date de fabrication            Data di produzione            Productiedatum</p>
	<p>Artikelnummer            Article number            Numéro d'article            Numero articolo            Artikelnummer</p>		<p>Achtung            Attention            A noter            Attenzione            Achtung</p>
	<p>einzelner Patient - mehrfach anwendbar            single patient - multiple use            patient individuel - utilisable plusieurs fois            singolo paziente - utilizzabile più volte            één patiënt - meerdere keren te gebruiken</p>		<p>Fertigungslosnummer, Charge            Production lot number, batch            Numéro de lot de fabrication, lot            Numero di lotto di produzione, partita            Productienummer, batch</p>
	<p>CE-Kennzeichnung            CE marking            Marquage CE            Marcatura CE            CE-markering</p>		<p>Medizinprodukt            Medical device            Dispositif médical            Dispositivo medico            Medisch hulpmiddel</p>
	<p>Eindeutige Gerätekenung            Unique device identifier            Identifiant unique de l'appareil            Identificazione univoca del dispositivo            Unieke apparaatidentificatie</p>		<p>Gebrauchsanweisung            Instructions for use            Mode d'emploi            Istruzioni per l'uso            Gebruiksaanwijzing</p>
	<p>Reinigung            Cleaning            Cleaning            Pulizia            Reiniging</p>		<p>Prothese            Prosthesis            Prothèse            Protesi            Prothese</p>

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## EN - Instruction Manual

Please read the IFU carefully before fitting.  
Only correct usage will warrant the function.  
The instructions for use are not intended for distribution to patients!

### 1. Intended Use

Adapters are load-bearing components within a prosthesis.

### 2. Technical data

- Rotation 360°
- Material: Titanium
- shortable pushbutton
- weight limit: 150 kg

#### REF

Order No.	Description	Additional Height	Weight	Article No.
TA-1	4-Hole connection	11 mm	220 g	4 162 010 00 00 000
TA-2	M36 x 1,5	11 mm	268 g	4 162 020 00 00 000

### 2.1 Individual and spare parts

#### REF

Order No.	Description	Article No.
E-TA01	Push Button Black	4 162 019 00 00 000
E-TA02	Release slider	4 162 019 00 00 001

Hex socket head screw:

#### REF

Order No.	Thread	Length	Units p. package	Article No.
M28-12	M8	12 mm	10	4 226 050 00 00 012
M28-16	M8	16 mm	10	4 226 050 00 00 016

### 3. Indications/ Contraindications

Indications:

- Leg amputation
- Dysmelia

Contraindications:

- None

### 4. Side effects

There are no known side effects.

## 5. General safety instructions

- This medical product is designed for single patient, multiple use.
- Fitting/ service of this medical device is only allowed by a certificated orthopedic professional.
- The patient must be instructed by the technician on correct use and informed about possible side effects/ residual risks.
- Improper modification or application to the product is not allowed. In case of non-observance, the function of the product may be impaired, so that product liability is excluded.
- Only approved screws may be used.
- The maximum weight loads are listed under "2. Technical Data". Additional loads must be considered when calculating the weight.
- Keep the adapter away from temperatures above 150°C. Do not use any equipment producing hot air while assembling the prosthesis.
- The patient must be advised that the adapter has to be snapped back after disengaging (clearly "click" tone)
- In case of an accident or any other extraneous cause occurring, service is strongly recommended. Damage parts must be replaced.
- These adapters are not designed for use in water.

## 6. Residual risks

Despite proper installation and use of the fitting parts in a prosthesis, adverse circumstances can lead to insecurity (e.g. stumbling, loss of balance, falling).

## 7. Usage and Installation

### 7.1 Function

Install the TA-1 or TA-2 above the prosthetic knee joint. The tibial section of prosthesis can then be rotated independent of the socket when the release button is pressed. The adaptor will automatically lock back when returned to the original position.

### 7.2 Installation

Install the rotation adapter TA-1 directly on the 4-hole attachment plate.

Install the TA-2 on a socket anchor adaptor with an internal M36 thread connector.

**An additional adaptor is not required.**

Tighten the included M6 screws for the TA-1 with a torque wrench to 12Nm.  
Use Loctite® 222 to secure the screws.

The TA-2 can be turned out at max. 270° to position it in the required position.

If the above-mentioned settings are overtightened, breakage of the housing may occur.

After positioning the turntable adaptor, tighten the securing cap screw of the lamination anchor to a torque of 8 Nm.

Secure with Loctite 222 when finishing the prosthesis.

The TA-1 or TA-2 has four adjusting screws distally and for attachment to the prosthetic knee joint.

### 7.3 Instructions of Use - Headless Screws

By using the adjusting screws M28-12 or M28-16, static corrections are possible at any time. If a module has to be replaced or removed the previously adjusted position is maintained if only two adjacent adjusting screws (not opposite), the ones screwed in at the deepest, are removed.

If the adjusting screws are too long (damage of the foam cover) or too short (reduced stability) these have to be replaced with appropriate ones.

Use a torque wrench to tighten the adjusting screws.  
Tightening torque: 15 Nm.

Secure the various screws, used for assembly, with Loctite 222.

### 8. Maintenance and Cleaning

The adapters must be checked as part of regular prosthetic service/ maintenance.

Soap and hand warm water  
Do not use aggressive solvents for cleaning.

### 9. CE-Conformity

The product satisfies the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council (MDR) and bears the CE mark. All major incidents related to the product need to be informed to Uniprox and the competence European Authority.

### 10. Warranty and Guarantee

We recommend change/ renewing the adapter after 5 years of usage.

Warranty is provided under the terms of sales and supply of Uniprox GmbH & Co. KG provided that the above conditions are met.

### 11. Storage and Disposal

This product has no special storage regulations. Please note the country-specific disposal.

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