Gebrauchsanweisung  
*Instruction Manual*  
Mode d'emploi  
*Istruzioni per l'uso*  
Gebruiksaanwijzing

**DAHO® Typ 4**  
**DAHO® Typ 4P**

Version: 5, Stand: 30.09.2025

Uniprox GmbH & Co. KG  
H.-Heine-Str. 4  
07937 Zeulenroda-Triebes

Tel. +49 (0) 36628-66-33 00  
Fax +49 (0) 36628-66-33 55  
E-Mail [info@uniprox.de](mailto:info@uniprox.de)

Symbole/ Icons/ Symboles/ Simboli/ Symbolen .....	3
DE - Gebrauchsanweisung.....	4
EN - Instruction Manual .....	7
FR - Mode d'emploi .....	10
IT - Istruzioni per l'uso .....	13
NL - Gebruiksaanwijzing .....	16

## Symbole/ Icons/ Symboles/ Simboli/ Symbolen

	<p>Hersteller            Manufacturer            Fabricant            Produttore            Fabrikant</p>		<p>Herstellungsdatum            Date of manufacture            Date de fabrication            Data di produzione            Productiedatum</p>
	<p>Artikelnummer            Article number            Numéro d'article            Numero articolo            Artikelnummer</p>		<p>Achtung            Attention            A noter            Attenzione            Achtung</p>
	<p>einzelner Patient - mehrfach anwendbar            single patient - multiple use            patient individuel - utilisable plusieurs fois            singolo paziente - utilizzabile più volte            één patiënt - meerdere keren te gebruiken</p>		<p>Fertigungslosnummer, Charge            Production lot number, batch            Numéro de lot de fabrication, lot            Numero di lotto di produzione, partita            Productienummer, batch</p>
	<p>CE-Kennzeichnung            CE marking            Marquage CE            Marcatura CE            CE-markering</p>		<p>Medizinprodukt            Medical device            Dispositif médical            Dispositivo medico            Medisch hulpmiddel</p>
	<p>Eindeutige Gerätekenung            Unique device identifier            Identifiant unique de l'appareil            Identificazione univoca del dispositivo            Unieke apparaatidentificatie</p>		<p>Gebrauchsanweisung            Instructions for use            Mode d'emploi            Istruzioni per l'uso            Gebruiksaanwijzing</p>
	<p>Reinigung            Cleaning            Cleaning            Pulizia            Reiniging</p>		<p>Immobilisierung            Immobilization            Immobilisation            Immobilizzazione            Immobilisatie</p>
	<p>Korrektur            Correction            Correction            Correzione            Correctie</p>		<p>Mobilisierung            Mobilization            Mobilisation            Mobilizzazione            Mobilisatie</p>

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## EN - Instruction Manual

Please read the IFU carefully before fitting.  
Only correct usage will warrant the function.

### 1. Intended Use

Hand orthoses are used for immobilization, mobilization and/ or correction.

### 2. Technical data

#### 2.1 DAHO® Typ 4

Support orthosis

**REF**

Order No.	Size	Side	Article No.
DAH04-SL	S	left	4 501 080 01 01 000
DAH04-ML	M	left	4 501 080 01 02 000
DAH04-LL	L	left	4 501 080 01 03 000
DAH04-XLL	XL	left	4 501 080 01 04 000
DAH04-SR	S	right	4 501 080 02 01 000
DAH04-MR	M	right	4 501 080 02 02 000
DAH04-LR	L	right	4 501 080 02 03 000
DAH04-XLR	XL	right	4 501 080 02 04 000

#### 2.2 DAHO® Typ 4P

Support orthosis perforated

**REF**

Order No.	Size	Side	Article No.
DAH04P-SL	S	left	4 501 090 01 01 000
DAH04P-ML	M	left	4 501 090 01 02 000
DAH04P-LL	L	left	4 501 090 01 03 000
DAH04P-SR	S	right	4 501 090 02 01 000
DAH04P-MR	M	right	4 501 090 02 02 000
DAH04P-LR	L	right	4 501 090 02 03 000

### 3. Indications/ Contraindications

Indications:

- In the primary phase of treatment, e.g. after burns
- With acute pain
- For physiological support after hand surgery

Contraindications:

- Extreme deformities of the forearm or hand
- Extreme small or large hands
- Open wounds (without wound coverage)

### 4. Side effects

If worn for a long time, contact and the corrective effect of the orthosis can cause marks and irritation on the skin.

### 5. General safety instructions

- This medical product is designed for single patient, multiple use.
- Fitting/ service of the medical device is only allowed by a certificated orthopedic professional.
- The patient must be instructed by the technician on correct use and informed about possible side effects/ residual risks.
- The DAHO® Handorthosis is a prescription product that should be worn according to the medical instructions.
- DAHO® Handorthosis should only be worn in accordance with these instructions and for the indications described.
- In the event of incorrect use product liability is excluded.
- In the event of over sensitivity of the skin, known contact allergies and sensitivity to pressure, a cotton glove should be worn under the DAHO® Handorthosis.
- No inexpert alterations may be made to the product. This may result in the performance of the product being limited, with the consequence that product liability is excluded.
- Patients should consult their doctor in the event of unusual changes in the hand function (e.g. an increase of the number of complaints).
- Combination with other products must be discussed with the patient 's doctor beforehand.
- In the event of changes in performance or malfunctions of the orthosis, the patient should contact their orthopedic technician or doctor immediately.
- Avoid contact of the orthosis with wounds and skin deformations, without wound coverage!

### 6. Residual risks

When using the hand orthosis (outside of clothing) it can get caught/ tangled with other objects.

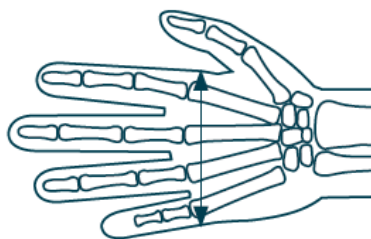
Despite proper use, skin irritation and pressure points can occur in individual cases.

## 7. Application information

### 7.1 Size Selection

To select the right orthotic size, the metacarpal width is measured at the level of the MCP joints.

Size	Midhand width
S	till 7 cm
M	till 8 cm
L	till 9 cm
XL	till 10 cm

### 7.2 Adjustment

The orthosis is worn on the skin or on a thin glove. According to the specific condition, the orthosis must be individually adapted either with thermoplastic or a mechanical conversion.

- After the orthosis has been adjusted the hook and loop fasteners (included) can be fitted.
- Type 4P is more breathable, because of the perforations in the device.

## 8. Maintenance and Cleaning

The orthosis can be washed with mild soap and hand warm water.

Commercially available disinfectants should be used for surface disinfection.

## 9. CE-Conformity

The product satisfies the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council (MDR) and bears the CE mark. All major incidents related to the product needs to be informed to Uniprox and the competence European Authority.

## 10. Warranty and Service life

We recommend to change/ renew the hand orthosis after 6 month.

Warranty is provided under the terms of sale and supply of Uniprox GmbH & Co. KG provided that the above conditions are met.

## 11. Storage and Disposal

The orthosis has no special storage regulations.

Please note the country-specific disposal.

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