
Gebrauchsanweisung für den Techniker
Instructions for use for technicians
Mode d'emploi pour le technicien
Istruzioni per l'uso per il tecnico
Gebruiksaanwijzing voor de technicus

F310 Uniprox Chopart
F340 Uniprox Mid-Profile
F360 Uniprox High-Profile

Version: 1, Stand: 18.06.2026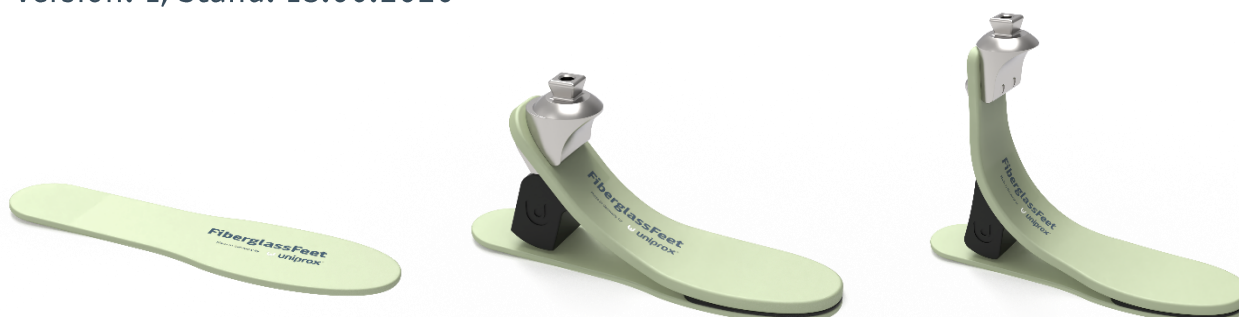

Uniprox GmbH & Co. KG
H.-Heine-Str. 4
07937 Zeulenroda-Triebes

Tel. +49 (0) 36628-66-33 00
Fax +49 (0) 36628-66-33 55
E-Mail info@uniprox.de

Symbole/ Icons/ Symboles/ Simboli/ Symbolen	3
DE - Gebrauchsanweisung.....	4
EN - Instruction Manual	8
FR - Mode d'emploi	12
IT - Istruzioni per l'uso	16
NL - Gebruiksaanwijzing	20

Symbole/ Icons/ Symboles/ Simboli/ Symbolen

	<p>Hersteller Manufacturer Fabricant Produttore Fabrikant</p>		<p>Herstellungsdatum Date of manufacture Date de fabrication Data di produzione Productiedatum</p>
	<p>Artikelnummer Article number Numéro d'article Numero articolo Artikelnummer</p>		<p>Achtung Attention A noter Attenzione Achtung</p>
	<p>Seriennummer Serial number Numéro de série Numero di serie Seriennummer</p>		<p>Fertigungslosnummer, Charge Production lot number, batch Numéro de lot de fabrication, lot Numero di lotto di produzione, partita Productielosnummer, batch</p>
	<p>CE-Kennzeichnung CE marking Marquage CE Marcatura CE CE-markering</p>		<p>Medizinprodukt Medical device Dispositif médical Dispositivo medico Medisch hulpmiddel</p>
	<p>Eindeutige Gerätekenung Unique device identifier Identifiant unique de l'appareil Identificazione univoca del dispositivo Unieke apparaatidentificatie</p>		<p>Gebrauchsanweisung Instructions for use Mode d'emploi Istruzioni per l'uso Gebruiksaanwijzing</p>
	<p>Reinigung Cleaning Cleaning Pulizia Reiniging</p>		<p>einzelner Patient - mehrfach anwendbar single patient - multiple use patient individuel - utilisable plusieurs fois singolo paziente - utilizzabile più volte één patiënt - meerdere keren te gebruiken</p>

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EN - Instruction Manual

Please read the IFU carefully before fitting.
Only correct usage will warrant the function.
The instructions for use are not intended for distribution to patients!

1. Intended Use

Prosthetic feet are the base structure of a lower limb prosthesis.

2. Technical data

2.1 F310 Uniprox Chopart

- Weight limit: 159 kg
- Degree of mobility: 1 - 4
- Robust fiberglass construction in 6 spring categories
- Material resistant to fresh water, salt water and chlorinated water
- Incl. foot cosmetics, Spectra sock and 2-component adhesive
- Heel height: 10 mm
- Installation height: approx. 15 mm (incl. foot cosmetics)

REF

Order No.	Sizes	Weight	Article No.
F310-Size/ Side/ Category	22 - 30	130 g (Size 26) without cosmetics	4 603 180 0X 0Y 0ZZ
X = Side: 1 = Left, 2 = Right	Y = Category	ZZ = Size	

2.2 F340 Uniprox Mid-Profile

- Weight limit: 159 kg
- Degree of mobility: 2 - 4
- Robust fiberglass construction in 6 spring categories
- Material resistant to fresh water, salt water and chlorinated water
- Heel height: 10 mm
- Installation height: approx. 118 mm (incl. foot cosmetics)

REF

Order No.	Sizes	Weight	Article No.
F340-Size/ Category	23 - 29	597 g (Size 26) without cosmetics	4 603 190 00 0Y 0ZZ
E-F80-Size/ Side	22 - 30	276 g (Size 26)	4 602 029 0X 00 0ZZ
X = Side: 1 = Left, 2 = Right	Y = Category	ZZ = Size	

2.3 F360 Uniprox High-Profile

- Weight limit: 159 kg
- Degree of mobility: 2 – 4
- Robust fiberglass construction in 6 spring categories
- Material resistant to fresh water, salt water and chlorinated water
- Heel height: 10 mm
- Installation height: approx. 162 mm (incl. foot cosmetics)

REF

Order No.	Sizes	Weight	Article No.
F360-Size/ Category	23 - 29	622 g (Size 26) without cosmetics	4 603 200 00 0Y 0ZZ
E-F80-Size/ Side	22 - 30	276 g (Size 26)	4 602 029 0X 00 0ZZ

X = Side: 1 = Left, 2 = Right Y = Category ZZ = Size

2.4 Category selection

2.4.1 F310 Uniprox Chopart

Category	Foot size	Low load hiking, golf	Medium load mountain hiking, jogging, skiing	High load tennis, football, basketball
1	22 – 27	33 – 64 kg	33 – 64 kg	-
2	22 – 28	65 – 79 kg	65 – 79 kg	33 – 64 kg
3	22 – 29	80 – 95 kg	80 – 95 kg	65 – 79 kg
4	23 – 30	96 – 111 kg	96 – 111 kg	80 – 95 kg
5	24 – 30	112 – 127 kg	112 – 127 kg	96 – 111 kg
6	25 - 30	128 – 159 kg	128 – 159 kg	112 – 127 kg

2.4.2 F340 Uniprox Mid-Profile/ F360 Uniprox High-Profile

Category	Foot size	Low load hiking, golf	Medium load mountain hiking, jogging, skiing	High load tennis, football, basketball
1	23 – 27	33 – 64 kg	33 – 64 kg	-
2	23 – 28	65 – 79 kg	65 – 79 kg	33 – 64 kg
3	23 – 29	80 – 95 kg	80 – 95 kg	65 – 79 kg
4	24 – 29	96 – 111 kg	96 – 111 kg	80 – 95 kg
5	25 – 29	112 – 127 kg	112 – 127 kg	96 – 111 kg
6	27 - 29	128 – 159 kg	128 – 159 kg	112 – 127 kg

Instructions for category selection

To optimize the selection and ensure the safety of the patient, follow the two steps to determine the appropriate category:

1. Find the column corresponding to the patient's load level.
2. Find the patient's weight in the selected column.

If the patient is carrying heavy loads or will reach a higher load level within a year, select the next higher category.

2.5 Individual and spare parts

REF

Order-No.	Description	Article No.
E-F80-ZZ X*	Foot cosmetics	4 602 029 0X 00 0ZZ
E-F60-SS	Spectra-Sock	4 603 019 00 39 000
M42-50	PLUSeries 1 Min. Composite, 50 ml	4 693 120 00 00 050
M42-220	PLUSeries 1 Min. Composite, 220 ml	4 693 120 00 00 220

ZZ = Size X = Side: 1 = Left, 2 = Right

3. Indications/ Contraindications

Indications:

- Leg amputation
- Dysmelia

Contraindications:

- None

4. Side effects

There are no known side effects.

5. General safety instructions

- This medical product is designed for single patient, multiple use.
- Fitting/ service of this medical device is only allowed by a certificated orthopedic professional.
- The patient must be instructed by the technician on correct use and informed about possible side effects/ residual risks.
- Improper modification or application to the product is not allowed. In case of non-observance, the function of the product may be impaired, so that product liability is excluded.
- The maximum weight loads are listed under "2. Technical Data". Additional loads must be taken into account when calculating the weight.
- During installation of the foot, skin contact with PU adhesive may occur. It must be ensured that the employees concerned have completed the legally required training on handling diisocyanates.
- Using the foot can cause fiber splinters in the edge area. Be careful when maintaining the foot to avoid injury.
- In case of functional impairments, abnormalities or external force (e.g. due to a fall), we recommend that you have the device checked by a specialist workshop.
- After contact with fresh water, salt water or chlorinated water, rinse your feet with water and allow them to dry thoroughly.


6. Residual risks

Despite proper installation and use of the fitting parts in a prosthesis, adverse circumstances can lead to insecurity (e.g. stumbling, loss of balance, falling).

7. Installation


7.1 F310 Uniprox Chopart

Position the socket on the sole plate as shown in the illustration, ensuring proper plantar/ dorsal flexion and external rotation. The rear third of the Chopart plate must be degreased and roughened before bonding. Fix the sole plate in the rear third with suitable epoxy or PU-adhesive for the trial. Fill the space between the socket and sole plate around the rear third with adhesive. For the final finish, the connection between the socket and the Chopart plate must be covered with enough reinforcing material around the rear third.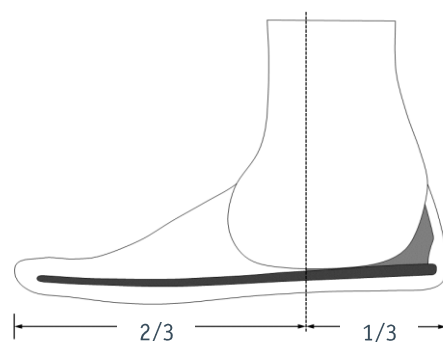

7.2 F340 Uniprox Mid-Profile/ F360 Uniprox High-Profile

To position the foot in the A/P line, align the foot so that the build line passes through the centre of the pyramid (see figure).

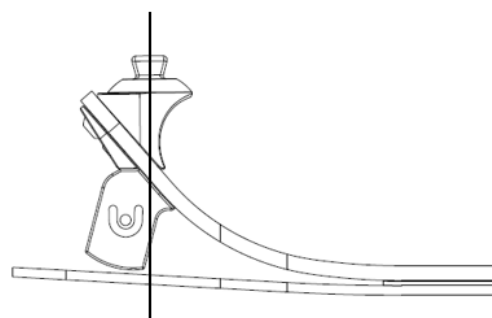8. Maintenance and cleaning

The function of the prosthetic foot must be checked at regular intervals of 6, maximum 12 months.

If there is a functional limitation that cannot be remedied by changing the wearing parts (Spectra sock or foot shell), the prosthetic foot must be sent to Uniprox GmbH & Co. KG for inspection.

The feet can be cleaned with soap and lukewarm water.
Do not use aggressive cleaning agents.

9. CE-Conformity

The product satisfies the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council (MDR) and bears the CE mark. All major incidents related to the product need to be informed to Uniprox and the competence European Authority.

10. Warranty and Guarantee

The service life is 2 million step cycles. The final assessment is made by the supervising technician, considering the user's activity. The warranty only applies under the conditions in accordance with the terms and conditions of sale and delivery (GTC) of Uniprox GmbH & Co. KG. A 2-year warranty is provided for material and manufacturing defects.

11. Storage and Disposal

There are no special storage requirements.
Please note the country-specific disposal.

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