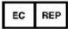
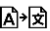
Symbol-Legende Icon Legend

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8. Maintenance and Cleaning

A 12 months check of the locking device is recommended. During the annual service, check and if necessary replace (depending on usage impact) the clamp discs (E-ShL61) and pin (E-ShL02, E-ShL03 or E-ShL54).

Cleaning:

Compressed Air up to 2 bar
Soap and hand warm water
Do not use aggressive solvents for cleaning

9. CE-Conformity

The product satisfies the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council (MDR) and bears the CE mark. All major incidents related to the product needs to be informed to Uniprox and the competence European Authority.

10. Warranty and Guarantee

Depending on the degree of usage, the locking device can generally be used for 5 years with regular maintenance. Warranty is provided under the terms of sale and supply of Uniprox GmbH & Co. KG provided that the above conditions are met.

11. Storage and Disposal

This product has no special storage regulations. Please note the country-specific disposal.

Please direct any questions to:

Customer Service: + 49 (0) 36628-66-33 70
Fax: + 49 (0) 36628-66-33 77
E-mail: info@uniprox.de


7. Installation

The shuttle lock ShL6A is inserted into a prepared cavity at the distal end of the socket and fixed with a 4-hole adapter.

For the release mechanism, a hole must be made on the desired side of the socket.

7.1 Inserting the release mechanism

1. Place the release mechanism in the correct position (Releaser tip upwards) and insert the opening of ShL6A.

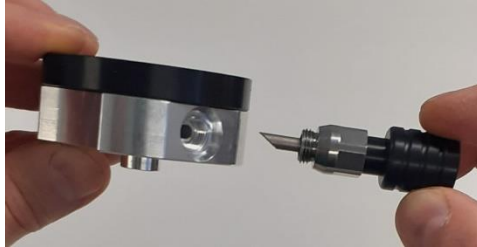2. When screwing in, keep the knob of the mechanism slightly pressed.

3. Insert the clamping spring and the clamping washers.

The beveled clamping disk must be on top so that the bevel is visible.

Take note that the washers are located under the hold-down screw and the unlocker tip.

Slight loosening of the unlocking screw makes it easier to thread in the discs.

Then tighten the screw connection with an open-end wrench while pressing the release button.

4. Attach the plastic funnel with the 2 included Phillips screws.

7.2 Shortening the release button

The plastic release button can be shortened from the outside to a maximum of the 3rd groove.

The release button must be secured with Loctite® 243.

Make sure that no Loctite penetrates the mechanism.

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Bestell-Nr.	Bezeichnung	Artikelnummer
E-ShL06	Druckknopf schwarz	4 147 019 00 00 006
E-ShL40	Mechanik ShL40A	4 147 149 00 00 004
E-ShL02	Pin, glatt, 55 mm	4 147 019 00 00 002
E-ShL03	Pin, glatt, 90 mm	4 147 019 00 00 003
E-ShL54	Pin, glatt, geriffelte Spitze	4 620 009 00 33 001
E-ShL61	Austausch-Set für ShL40A	4 655 049 00 06 000

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5. General Instructions

- This medical device is designed for single patient, multiple use.
- Fitting/ service of this medical device is only allowed by a certificated orthopedic professional.
- The patient must be instructed by the technician on correct use and informed about possible side effects/ residual risks.
- The patient should be instructed to don the liner so, that the distal pin is correctly aligned. If the pin is not aligned correctly, this could cause unsafe attachment to the locking device and/ or a jammed pin, what will be difficult to release, or that the pin will not be securely locked in the mechanism.
- We recommend an installation and function check by a specialist in case of malfunction, abnormalities such as external forces.
- Improper modification or application to the product is not allowed. In case of non-observance, the function of the product may be impaired, so that product liability is excluded.
- Avoid exposing the product to corrosives such as salt water, chlorinated water, ammonia, highly acidic, or highly alkaline agents. In case of exposure, rinse with fresh water and allow to thoroughly dry.

6. Residual risks

If the liner is applied inaccurately (incorrect alignment of the pin), locking problems can occur with the closure piece. Despite proper installation and use of the fitting parts in a prosthesis, adverse circumstances can lead to insecurity (e.g. stumbling, loss of balance, falling).

2.2 Individual and spare parts

Order No.	Description	Article No.
E-Shl06	Push button black	4 147 019 00 00 006
E-ShL40	Mechanism	4 147 149 00 00 004
E-ShL02	Pin, smooth 55 mm	4 147 019 00 00 002
E-ShL03	Pin, smooth, 90 mm	4 147 019 00 00 003
E-ShL54	Pin, smooth with grooved tip	4 620 009 00 33 001
E-ShL61	Replacement kit for ShL40A	4 655 049 00 05 000

2.3 Adapter options

Direct connection optional by the following socket adapters: A10, A11, A13 or A14.

3. Indications/ Contraindications

Indications:

- Leg amputation
- Dysmelia

Contraindications:

- sensitive distal end of residual limb

4. Side effects

Irritation at the end of the stump due to increased tensile stress.

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Please read the IFU carefully before fitting. Only correct usage will warrant the function.

1. Intended Use

Socket fixations are used to securely connect the residual limb in the socket of a leg prosthesis.

2. Technical data

Locking: with smooth pin
Weight limit: max. body weight 150 kg

Order No.	Material	Weight	Installation height	Article No.
ShL6A	Aluminum, plastic	169 g	22 mm	4 655 060 00 01 000

- English -

2.1 Scope of delivery

- Locking device
- Pin, smooth, 55 mm with corrugated tip